



# Faults on the EasyPull

If problems or faults occur on the EasyPull, refer to the Chapter "Faults and Troubleshooting" in the full Operating Instructions. Contact our Customer Service Department if a fault can not be rectified using the information given there.

# **Customer Service**

The AMF-Bruns customer service department will be more than pleased to assist in questions regarding the ordering of spare parts, fault rectification, maintenance and repair work.

Address: AMF-Bruns GmbH & Co. KG

Hauptstraße 101 26689 Apen

Telephone: +49 (0) 44 89 / 72 72-30

Telefax: +49 (0) 44 89 / 62 45

E-mail: service.hubmatik@amf-bruns.de

AMF-Bruns

### **BRIEF OPERATING INSTRUCTIONS**

EasyPull



These brief operating instructions do not in any way replace the full Operating Instructions! Please read the full Operating Instructions thoroughly before using the product for the first time!



# AMF-Bruns GmbH & Co. KG

Hauptstraße 101 | 26689 Apen | Germany Telephone +49 (0) 4489 / 72 72 30 Telefax +49 (0) 44 89 / 6245 service.hubmatik@amf-bruns.de www.amf-bruns.de




Note: If the EasyPull is used to transport a wheelchair that is different to the one previously transported, the EasyPull must be re-adjusted as described in the Operating Instructions.

Warning! The occupied or empty wheelchair can move in an uncontrolled manner when it is pulled into the vehicle by the EasyPull. Guide the occupied or empty wheelchair by hand until it has reached its foreseen position in the vehicle.

#### Loading a Passenger / Wheelchair

- 1. Fold the access ramp out.
- 2. Accompany the occupied or empty wheelchair to the bottom of the access ramp.
- 3. Apply the wheelchair's brakes.
- 4. Take the EasyPull belts up with one hand.
- 5. Press and hold push-button "C" on the remote control to reel the belts out quickly.

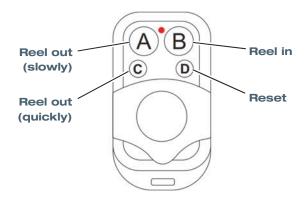

A beep sounds.

- 6. Pull the EasyPull belts out to the wheelchair.
- 7. Release the push-button.

The beeping sound stops.

8. Attach the belts to the front hitching point adapters on the wheelchair. Use the belt loops if no hitching point adapters are available.





- 9. Release the wheelchair's brakes.
- 10. Take up a position behind the wheelchair and keep a firm hold of it by the handles.
- 11. Press and hold push-button "B" on the remote control.

An intermittent beep sounds.

12. Guide the wheelchair whilst it is being pulled into the rear of the vehicle. Make sure that the belts are always lightly tensioned when doing so.

When the wheelchair has reached its foreseen position in the vehicle, the EasyPull will stop and the latch will be applied to the winch.

A continuous beep sounds.

- 13. Try to pull the wheelchair backwards to make certain that the latch has been applied to the winch.
- 14. Release push-button "B".

The beeping sound stops.

- 15. Apply the wheelchair's brakes.
- 16. Secure the rear of the wheelchair using the belts provided.
- i Note: If the EasyPull has not stopped and latched in position or if the rear of the wheelchair can not be secured with the rear belts, the EasyPull must be re-adjusted as described in the Operating Instructions.
- 17. Secure the passenger with a seat belt.
- 18. Fold the access ramp up.
- 19. Close the rear doors / tailgate.

#### **BRIEF OPERATING INSTRUCTIONS**

EasyPull

# Disembarking a Passenger / Unloading a Wheelchair

- Fold the access ramp out.
- 2. Release the passenger's seat belt.
- Detach and remove the rear belts securing the rear of the wheelchair.
- 4. Release the wheelchair's brakes.
- Take up a position behind the wheelchair and keep a firm hold of it by the handles.
- 6. Press and hold push-button "A" on the remote control.

A beep sounds.

- Accompany the passenger / wheelchair out of the vehicle.
- 8. Remain in position behind the wheelchair until it has left the access ramp completely.
- 9. Release the push-button.

The beeping sound stops.

- 10. Apply the wheelchair's brakes.
- 11. Detach the EasyPull belts from the wheelchair.
- 12. Hold the belts up with one hand.
- 13. Press and hold push-button "B" on the remote control.

An intermittent beep sounds.

The EasyPull reels the belts into the vehicle.

- 14. Keep the belts under tension whilst they are being reeled into the vehicle.
- 15. Make certain that the belts are not twisted.

The winch stops and the latch is applied when at least one magnet reaches the winch housing.

A continuous beep sounds.

16. Release push-button "B".

The beeping sound stops.

- 17. Fold the access ramp up.
- 18. Close the rear doors / tailgate.

