
FUTURE**SAFE**

The optimum seating position

- The upper edge of the headrest is at the same height as the top of the passenger's head.
- The space between head and headrest is as small as possible.
- 3 The upper edge of the backrest is at shoulder height.
- The back rests on the backrest.

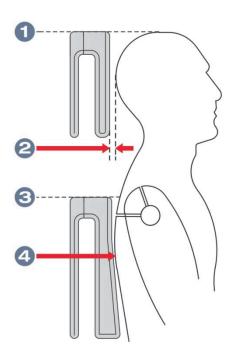

Note:

Persons who are dependent upon a wheelchair are not always in a position to assume a posture as shown in the above diagram.

Always try to find a balance between maximum protection and maximum comfort of the passenger.

The brief Operating Instructions are not a replacement for the full Operating Instructions! Please read the full Operating Instructions thoroughly before using the product for the first time!



AMF-Bruns GmbH & Co. KG

Hauptstraße | 26689 Apen | Germany Telephone +49 (0) 4489 / 72 72 30 Telefax +49 (0) 44 89 / 62 45 service.hubmatik@amf-bruns.de www.amf-bruns.de AMF-Bruns

BRIEF OPERATING INSTRUCTIONS

FUTURE**SAFE**



Pivoting the swivel arms

Press the operating lever on the swivel arm upwards.
This lifts the swivel arm out of the latch.




- 2. Pivot the swivel arm by 90° or 180° in the desired direction.
- 3. Release the operating lever.

This lowers the swivel arm, which once again engages in the latch.

Lengthening and shortening the swivel arms

The swivel arms can be pulled out (lengthened) and pushed in (shortened) without further preparation. They are kept in position by a perceptible latch.

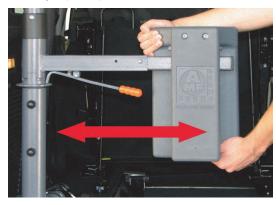

Adjusting the height of the head- and backrest



- 2. Move the head- or backrest to the desired height.
- 3. Release the latch button.

Note:

The front of the backrest has a slanting profile. This profile allows the backrest to be finely adjusted forwards or backwards by adjusting it's height.

Attention!

For safe use of the FutureSafe, the restraint systems for wheelchair and passenger must conform with the applicable local standards.


If in doubt, contact AMF-Bruns GmbH, to find out whether the restraint systems fitted to your vehicle are the correct systems.

BRIEF OPERATING INSTRUCTIONS

FUTURESAFE

Moving the FutureSafe with side anchorage plate forwards or backwards

- 1. Turn the red locking levers on both of the Smartfloor rails for the side anchorage plate to the vertical position.
- Loosen the securing screws by turning the black star grips.
- 3. Push the FutureSafe forwards or backwards to the desired position.

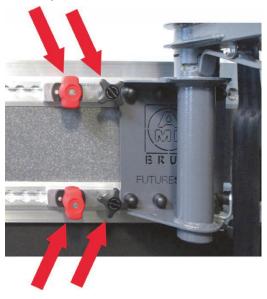

- Turn the red locking levers back to the horizontal position.
- Push or pull the FutureSafe gently forwards and backwards until it latches in position in the Smartfloor rails.
- Tighten the securing screws once again by turning the star grips.

Marning!

The head- and backrest of the FutureSafe only offer safety if the passenger is restrained by the seat belts and the wheelchair is secured in the vehicle.

Secure the wheelchair in the vehicle using the restraint systems provided. Always restrain the passenger with both the diagonal shoulder and lap belts.

Do not lay the lap belt over the armrests of the wheelchair. The lap belt must lie across the lap of the passenger. If the belt lies across the stomach, it is too high.

