5. Embarking a Passenger

If no passenger is to be embarked: Continue at Section 6.

- ⇒ Press and hold the "Lower" push-button as long as necessary to lower the platform onto the floor of the vehicle.
- Lower the narrow roll-off guard by operating the footoperated lever.
- ⇒ Release the wheelchair parking brake and switch an electrical wheelchair ON.
- Accompany the passenger off the platform into the vehicle.
- ⇒ Secure the passenger and wheelchair using the safety and restraint systems provided for this purpose.
- ⇒ Raise the narrow roll-off guard.
- ⇒ Press and hold the "Raise" push-button as long as necessary to raise the platform to the upper limit position.

6. Moving the Swivel Lift to the Driving Position

- Lower the narrow roll-off guard by operating the footoperated lever.
- ⇒ Fold the side guard rail down.
- ⇒ Fold the platform up, until it latches in position.
- ⇒ Release the main swivelling column latch using the unlatching lever.
- ⇒ Swivel the lift into the vehicle until it latches in position.
- Press the lever on the arrestor clamp downwards to the limit.
- Switch the Lift OFF at the master switch.

7. Emergency Mode

Raise:

⇒ Move the manually-operated pump operating lever up and down until the platform has been raised the desired position.

Lower:

- ⇒ Unscrew the operating lever from the manually-operated pump.
- ⇒ Remove the cladding.
- ⇒ Press the emergency lowering valve downwards and turn it slowly in an anti-clockwise direction.
- When the platform has reached the desired position, press the emergency lowering valve downwards and turn it as far as it will go in a clockwise direction.

Problems with the Swivel Lift

If problems or faults occur, refer to the Chapter "Faults and Troubleshooting" in the full Operating Instructions. Contact our Customer Service Department if a fault cannot be rectified using the information given there.

These brief operating instructions do not in any way replace the full Operating Instructions! Please read the full Operating Instructions thoroughly before using the product for the first time!

Subject to technical modifications.



AMF-Bruns GmbH & Co. KG

Hauptstraße | 26689 Apen | Germany Telephone +49 (0) 4489 / 72 72 22 Telefax +49 (0) 4489 / 62 45 service.hubmatik@amf-bruns.de www.amf-bruns.de **Brief Operating Instructions**

SWIVELLIFT BSL 350



Keep these Brief Operating Instructions close to hand in the vehicle!

Read the Operating Instructions thoroughly before using the ramp!



BRIEF OPERATING INSTRUCTIONS

Swivel Lift BSL 350

1. Swivelling Out to the 90° Position

- Switch the Lift ON at the master switch.
- Release the arrestor clamp.
- Release the main swivelling column latch using the unlatching
- Swivel the lift 90° out of the vehicle until it latches in position.
- Press the platform unlatching push-button.
- ⇒ Fold the platform down.
- Raise the side quard rail.
- ⇒ Raise the narrow roll-off guard.

2. Disembarking a Passenger

If no passenger is to be embarked: Continue at Section 3.


- Press and hold the "Lower" push-button as long as necessary to lower the platform onto the floor of the vehicle.
- ⇒ Lower the narrow roll-off guard by operating the footoperated lever.
- Release the safety and restraint systems from the passenger and wheelchair.
- Accompany the passenger onto the platform.
- Apply the wheelchair parking brake and switch an electrical wheelchair OFF.
- Raise the narrow roll-off quard.
- Press and hold the "Raise" push-button as long as necessary to raise the platform to the upper limit position.


3. Lowering the Platform

- Release the secondary swivelling column latch using the unlatching lever.
- ⇒ Swivel the lift a further 90° out of the vehicle until it latches in position.
- Press and hold the "Lower" push-button as long as necessary to lower the platform onto the ground.
- ⇒ Lower the wide roll-off guard by operating the foot-operated
- Pelease the wheelchair parking brake and switch an electrical wheelchair ON.
- Accompany the passenger off the platform.

4. Raising the Platform

- Accompany the passenger onto the platform.
- Apply the wheelchair parking brake and switch an electrical wheelchair OFF.
- Raise the wide roll-off quard.
- Press and hold the "Raise" push-button as long as necessary to raise the platform to the upper limit position.
- Release the secondary swivelling column latch using the unlatching lever.
- ⇒ Swivel the lift 90° into the vehicle until it latches in position.

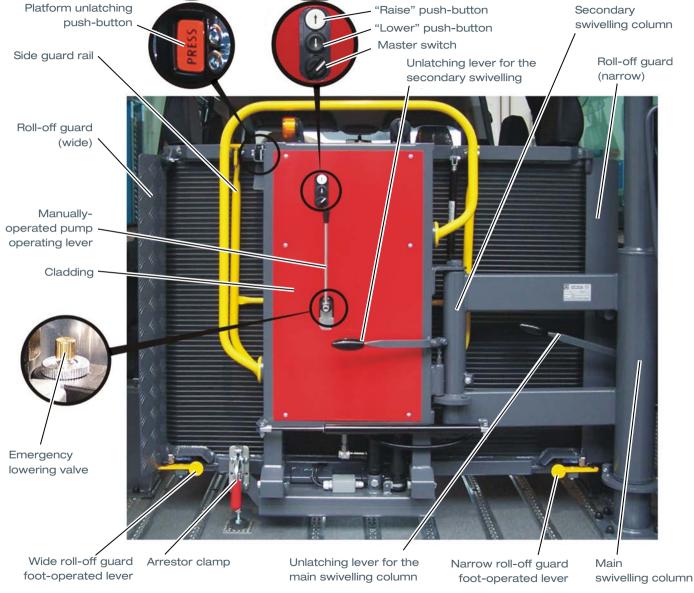

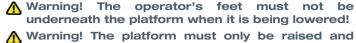

lowered when both roll-off guards are folded up!

or swivelled when an electrical wheelchair is switched OFF and / or the parking brake is applied!

 Marning! Only use the emergency mode to prevent an emergency situation for the passenger!



