#### AMF-Bruns

### **BRIEF OPERATING INSTRUCTIONS**

Opel Vivaro / Renault Trafic / Fiat Talento with Easy- and Easy-Flex-Ramp



## Using the Vehicle without a Passenger

After disembarking a passenger / unloading a wheelchair:

- 1. Fold the access ramp up until it audibly engages in the vertical position.
- 2. For the Easy-Flex-Ramp: Push the operating lever to the right, to fold the access ramp away.



- Fully fold the access ramp down inside the vehicle, until its bearing surfaces rest on the vehicle's floor at the sides of the floor section.
- Attention! Max. permissible access ramp load when deployed 300 kg.

  Max. permissible access ramp load when fully folded away 100 kg.

## Removing the seat retainers

In the interests of safety and comfort for the wheelchair passenger's feet, the seat retainers must be removed from a vehicle with a long wheelbase.



- 1. Lift the securing clip on a seat retainer.
- 2. Pull the seat retainer towards the rear.
- 3. Remove the seat retainer.
- 4. Repeat the procedure for the second seat retainer.

These brief operating instructions do not in any way replace the full Operating Instructions! Please read the full Operating Instructions thoroughly before using the product for the first time!



### AMF-Bruns GmbH & Co. KG

Hauptstraße 101 | 26689 Apen | Germany Telephone +49 (0) 4489 / 72 72 30 Telefax +49 (0) 4489 / 62 45 service.hubmatik@amf-bruns.de www.amf-bruns.de

# **Deploying the Access Ramp**

1. For the Easy-Flex-Ramp: Lift the access ramp up to the vertical position until it audibly engages.



Push the operating lever to the left, to deploy the access ramp.



3. Fold the access ramp out.



Caution! Keep a firm hold of the access ramp until it is fully in contact with the ground. Allow the access ramp to deploy slowly.

For a long wheelbase vehicle:

- 1. Fold the third row of seats up.
- 2. If necessary, remove the third row of seats.
- 3. Remove the seat retainers (see Figure overleaf).

# Loading a Passenger / Wheelchair

- Accompany the wheelchair to a position in front of the access ramp.
- 2. Operate the rocker switch such that the indicator lamp in the rocker switch lights up.






- 3. Pull the electrically operated belts out to the wheelchair.
- 4. Attach the electrically operated belts to the wheelchair.



5. Operate the rocker switch such that the indicator lamp in the rocker switch goes out.

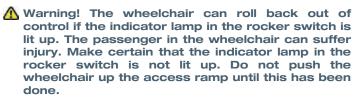

6. Push the wheelchair into the vehicle.

### **BRIEF OPERATING INSTRUCTIONS**

Easy-Ramp and Easy-Flex-Ramp

# Securing the Passenger and Wheelchair in the Vehicle


- 1. Clip the retractors onto the fittings.
- 2. Attach the retractor belts to the wheelchair.



- 3. Tension the retractor belts by tightening the belt reels.
- 4. Apply the wheelchair's parking brakes.
- Secure the passenger in the wheelchair using the restraint systems provided.
- 6. Fold the access ramp up, until it audibly engages.

# Disembarking a Passenger / Unloading a Wheelchair

- 1. Deploy the access ramp slowly.
- 2. Release the retractor belt reels.
- 3. Disconnect the retractors from the fittings.
- 4. Detach the retractor belts from the wheelchair.
- 5. Release the wheelchair's parking brakes.

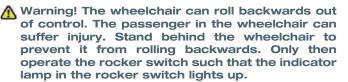

- Operate the rocker switch such that the indicator lamp in the rocker switch lights up.
- 7. Accompany the wheelchair down from the access ramp.
- 8. Detach the electrically operated belts from the wheelchair.
- 9. Fold the access ramp up, until it audibly engages.

