

Easy-Ramp and Easy-Flex-Ramp

Using the Vehicle Without a Passenger

After disembarking a passenger / unloading a wheelchair:

1. Fold the access ramp up until it audibly engages in the vertical position.
2. For the Easy-Flex-Ramp: Push the operating lever to the right, to fold the access ramp away.



3. Fully fold the access ramp down inside the vehicle, until its bearing surfaces rest on the vehicle's floor at the sides of the floor section.

**⚠ Attention! Max. permissible wheelchair access ramp load when deployed 300 kg.
Max. permissible access ramp load when fully folded away 100 kg.**

Faults

If problems or faults occur on the access ramp, refer to the Chapter "Faults and Troubleshooting" in the full Operating Instructions.

Contact our Customer Service Department if a fault can not be rectified using the information given there.

Customer Service

The AMF-Brunn customer service department will be more than pleased to assist in questions regarding ordering spare parts, fault rectification, maintenance and repair work.

Address: AMF-Brunn GmbH & Co. KG
Hauptstraße 101
26689 Apen

Telephone: +49 (0) 44 89 / 72 72 30

Telefax: +49 (0) 44 89 / 62 45


E-mail: service.hubmatik@amf-bruns.de

Internet: www.amf-bruns.de

AMF-Brunn

BRIEF OPERATING INSTRUCTIONS

Renault Kangoo / Mercedes-Benz Citan with Easy and Easy-Flex Ramp

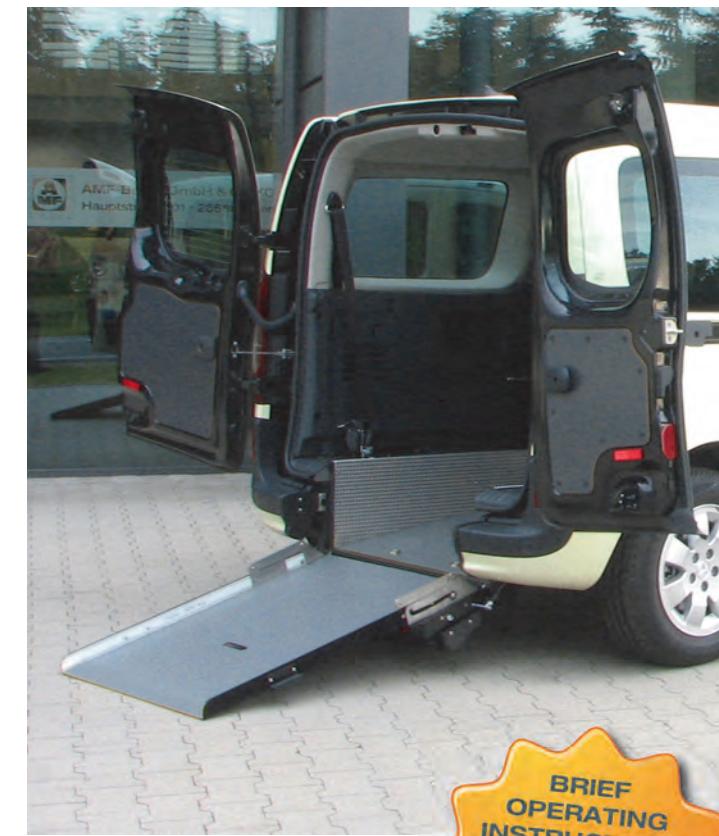

These brief operating instructions do not in any way replace the full Operating Instructions! Please read the full Operating Instructions thoroughly before using the product for the first time!

AMF-Brunn GmbH & Co. KG
Hauptstraße 101 | 26689 Apen | Germany
Telephone +49 (0) 4489 / 72 72 30
Telefax +49 (0) 4489 / 62 45
service.hubmatik@amf-bruns.de
www.amf-bruns.de

Art.-No.: 10022217 | 04/2017

BRIEF OPERATING INSTRUCTIONS
Read thoroughly before use,
keep readily available in the vehicle!

AMF-BRUNN
www.amf-bruns.de

BRIEF OPERATING INSTRUCTIONS

Easy-Ramp and Easy-Flex-Ramp

Deploying the Access Ramp

1. For the Easy-Flex-Ramp: Lift the access ramp up to the vertical position until it audibly engages.



2. Push the operating lever to the left, to deploy the access ramp.



3. Fold the access ramp out.



Caution! Keep a firm hold of the access ramp until it is fully in contact with the ground. Allow the access ramp to deploy slowly.

Loading a Passenger / Wheelchair

1. Secure the rear seats with the standard supports.
2. Accompany the wheelchair to a position in front of the access ramp.
3. Operate the rocker switch such that the indicator lamp in the rocker switch lights up.



4. Pull the electrically operated belts out to the wheelchair.
5. Attach the electrically operated belts to the wheelchair.



6. Operate the rocker switch such that the indicator lamp in the rocker switch goes out.

⚠ Warning! The wheelchair can roll back out of control if the indicator lamp in the rocker switch is lit up. The passenger in the wheelchair can suffer injury. Make certain that the indicator lamp in the rocker switch is not lit up. Push or drive the wheelchair up the access ramp only when this has been done.

7. Push or drive the wheelchair into the vehicle.
8. Switch the motor of an electrically driven wheelchair OFF.

BRIEF OPERATING INSTRUCTIONS

Easy-Ramp and Easy-Flex-Ramp

Securing the Passenger and Wheelchair in the Vehicle

1. Attach the retractor belts to the wheelchair.
2. Clip the retractors onto the fittings.



3. Tension the retractor belts by tightening the belt reels.
4. Apply the wheelchair's brake(s).
5. Secure the passenger in the wheelchair using the restraint systems provided.
6. Fold the access ramp up, until it audibly engages.

Disembarking a Passenger / Unloading a Wheelchair

1. Fold the access ramp out and slowly lower it to the ground.
2. Release the retractor belt reels.
3. Disconnect the retractors from the fittings.
4. Detach the retractor belts from the wheelchair.
5. Switch the motor of an electrically driven wheelchair ON.
6. Release the wheelchair's brake(s).

⚠ Warning! The wheelchair can roll backwards out of control. The passenger in the wheelchair can suffer injury. Stand behind the wheelchair to prevent it from rolling backwards. Only then operate the rocker switch such that the indicator lamp in the rocker switch lights up.

7. Operate the rocker switch such that the indicator lamp in the rocker switch lights up.
8. Accompany the wheelchair down from the access ramp.
9. Detach the electrically operated belts from the wheelchair.
10. Fold the access ramp up, until it audibly engages.